

A.M.I.® Disposable Instruments

Product Group Laparoscopy | Issue 10/2011



> A.M.I. TissueBags

- Extremely durable and tear-resistant extraction bags
- Open automatically in the abdominal cavity
- A.M.I. TissueBag Premium can remain in the abdomen until surgery is completed



> A.M.I. FasciaCutter Disposable Instrument

- Facilitates enlargement of trocar incisions
- Safe and controlled, even with the extraction bag still in the incision

A.M.I.® Disposable Instruments


A.M.I.'s extraction bags are fully-disposable instruments for removing specimen (e.g. appendix, gallbladders, ovaries etc.) during minimally-invasive surgical procedures. Special design features facilitate insertion into the abdominal cavity, as well as allowing easy, safe tissue retrieval. The A.M.I. FasciaCutter can be used to enlarge the trocar incision in a controlled manner, should more space be required for removal.

The extremely tear-resistant polyurethane bag:

- is covered by a polypropylene protection sleeve for insertion and fits through any 10 mm trocar
- is equipped with a metal ring which opens automatically once fully inserted in the abdomen
- remains open automatically until filling is completed, and
- has a tapered design which ensures lengthwise compression of specimen for easier removal.

A.M.I. TissueBag Premium

With this superior-quality, easy-to-handle extraction bag, you have the choice. Once filled with specimen, the bag can either be taken out immediately or left in the abdomen during the operation for later removal in four simple steps:

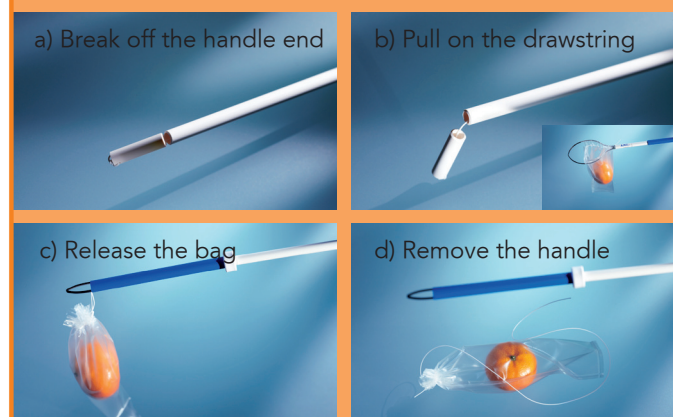

A.M.I. TissueBag

A particularly durable and easy-to-use extraction bag, which is removed from the abdominal cavity immediately after filling.



Order Code	A.M.I. Disposable Instruments	Technical Details
TBI1011	A.M.I. TissueBag Premium - For immediate removal of specimen, or - for removal on completion of surgery. The bag may be left in the abdomen during the procedure.	Opens 65 mm in diameter Max. filling volume: 210 ml Instrument length: 515 mm Delivered sterile, 10 instruments/box
TBI 100	A.M.I. TissueBag	Opens 65 mm in diameter Max. filling volume: 210 ml
TBI1021	A.M.I. TissueBag Large - For immediate removal of specimen.	Opens 80 mm in diameter Max. filling volume: 800 ml Instrument length: 485 mm Delivered sterile, 10 instruments/box
AFC 100	A.M.I. FasciaCutter Disposable Instrument - For the safe, controlled enlargement of trocar incisions.	Delivered sterile 20 instruments/box

A.M.I. Headquarters:

A.M.I. GmbH
Im Letten 1
6800 Feldkirch . Austria
t +43 5522 90505-0
f +43 5522 90505-4006
e info@ami.at
www.ami.at